


NEB XP Rechargeable Battery Pack #40-177-012

Instructions for Use:

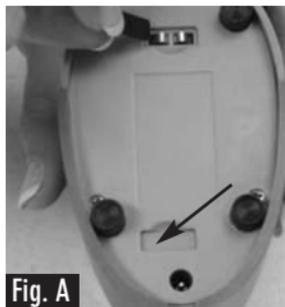


To charge the battery:

Insert the AC adapter plug (AC adapter is included with NEB XP Compressors, Model 40-135-000 and 40-136-000) into the receptacle of the battery pack (Fig. A). Following general safety precautions, plug the AC adapter into a standard 110 outlet.

Approximate charging times:

1st time use: 24 hours

Thereafter: 4-12 hours

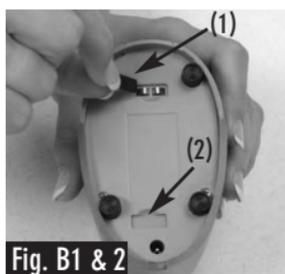

To use the battery:

Expose the battery terminal on the bottom of the compressor by removing the small, rubber battery terminal cover (Fig. B1). Reposition the cover over the small indented area (Fig. B2), located across from the battery terminal.

Attach the compressor and battery pack making sure that the battery terminals meet (Fig. C). The two components should snap together into place. Check that the compressor and battery pack are securely connected.



Proceed with your treatment as described in the instruction guidebook provided with your NEB XP compressor.



To unlock the battery:

While holding both the compressor and battery pack, press the rectangular-shaped button on the front of the battery pack and the components should release (Fig. D).

Reattach the battery terminal cover over the battery terminal.



Troubleshooting

If the battery pack and compressor do not securely connect and/or if the unit does not operate when the POWER button is pressed, (1) make sure that the battery terminal cover is not positioned over the battery terminal, and (2) that the battery is charged (green light is illuminated) (Fig. E).